



 What is PQtap?

It is a British designed app to record the flow and volume of your pee on an ongoing basis.

 Do I need a particular mobile device?

You will need a smartphone (or tablet) that enables you to download and view apps.

 Is it available on IOS and Android?

Yes, it is. Simply search for PQtap.

 How do I contact you if I have a suggestion, an issue, or further questions?

E-mail info@pqtap.com and we will get back to you as quickly as possible.

 What equipment do I need?

All you need is a smartphone to download the PQtap app and a suitable measuring jug.

 What type of jug is best?

You need a transparent jug and it is essential that it is marked in 50ml increments. You need to be able to see these measurements as you pee. Some people use a kitchen jug and simply mark the 50ml increments on the outside using a permanent marker. Others choose to purchase a jug that can be viewed from above or with large digits.

 Is it safe?

Yes, it is completely safe. You only pee into a jug while using the app.

 How is PQtap different to other similar apps?

Other apps focus on volume. This app also provides volume data, but the focus is on flow.

 Who uses PQtap?

Anyone with a medical condition that requires their urine flow rate to be monitored.

 How can I share the information with my clinician?

You can show your clinician on your phone screen when you meet, you can save the charts as a screenshot on your camera roll, or you can securely email the details via the app.

 Why don't I just do all this when I have an appointment at the clinic?

By doing this at home it is more convenient and comfortable for you. You do it when it suits you and over a period of time you build up some very useful data for your clinician. At the clinic it is on-demand, provides one-off data and it is more impersonal.

 Are my personal details safe?

Yes. We do not store your information. That is kept on your phone and you choose who can see these details.

